

# ACRAGUN ACRACLIP SCALP CLIP SYSTEM

Easy to use, Reliable, and  
Economical

ACRA-CUT introduces  
a clip gun system that fulfills  
the promise of fast, trouble-free  
application of scalp clips.


Its unique design prevents  
accidental clip ejection or  
failures to feed. The result  
makes the opening procedure  
faster and easier.

- ACRAGUN fits comfortably in any size hand.
- Apply clips at any angle, even upside-down, with complete control.
  - Combines the economy of a reusable gun with the convenience of sterile, preloaded cartridges.
  - Cartridges snap into the gun quickly and easily.

(Shown Actual Size)



ACRAGUN  
Comfortable and easy to use

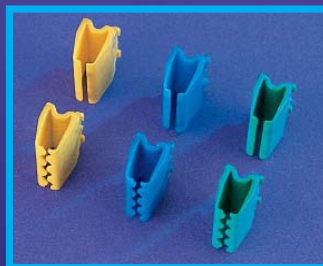

ACRACLIP  
Choice of three clamping forces  
and two jaw designs

## ACRA CUT®

989 Main St., Acton, MA 01720

[www.acracut.com](http://www.acracut.com)

Call: 800/227-2288

In MA: 978/263-0250 • Fax: 978/263-4102

U.S. Patent 5,665,097. Other patents pending.

## ACRAGUN SCALP CLIP SYSTEM

ACRA-CUT has developed a clip gun system that fulfills the promise of fast, trouble-free application of scalp clips.


Its unique design prevents accidental clip ejection or failures to feed. The result makes the opening procedure faster and easier.

### UNIQUE BENEFITS

- ACRAGUN fits comfortably in any size hand.
- ACRAGUN applies clips quickly at any angle, even upside down, with complete control.
- ACRAGUN has only four moving parts for maximum reliability.
- Combines the economy of a reusable gun with the convenience of sterile, preloaded cartridges.
- Cartridges snap and lock into the gun quickly and easily.
- Integral cartridge lock eliminates uncontrolled clip ejection.
- ACRACLIPs are available with serrated or straight jaws in a choice of three clamping forces.



### ACRACLIPs

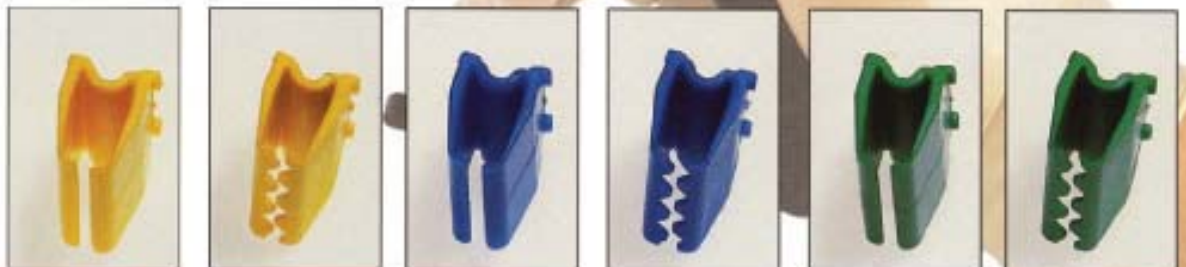

**Clamping Force** Light/Pediatric  
**Jaw Shape** Straight

Light/Pediatric  
Serrated

Medium  
Straight

Medium  
Serrated

Heavy  
Straight

Heavy  
Serrated

Color coded ACRACLIPs are available in two different jaw shapes and three clamping forces. Medium clamping force is equal to other commonly used scalp clips. Maximum clip opening is the same size as other scalp clips.

## ACRAGUN IS EASY TO LOAD



1. Snap the preloaded sterile cartridge into the ACRAGUN.



2. Lock the cartridge in place and the first clip slides into position, ready to use.

## ACRAGUN IS EASY TO USE



1. Squeeze the trigger to open the clip.



2. Apply the clip to the scalp.



3. ACRAGUN can apply clips at any angle - even upside down.



4. Release the trigger and the clip closes. The next clip slides into position ready to apply.



ACRAGUN comes with a protective sterilizing storage case.  
Catalog No. 500-100



ACRACLIPs come in single cartridge or double cartridge sterile blisterpacks. Each cartridge contains 15 clips.  
See chart at left

### ACRAGUN

Catalog No.

500-100 Reusable ACRAGUN with Sterilizing Case

500-135 Reusable ACRA-CUT Clip Applier/Remover

### ACRACLIP CARTRIDGES

#### TWO CARTRIDGE PACK, 15 clips per cartridge

Catalog No.	No. of Cartridges	Jaw Description	Clamping Force	Color	Application
500-101	Two (2)	Straight Edge	Light	Yellow	Pediatric Scalp
500-102	Two (2)	Straight Edge	Medium	Blue	Adult Scalp
500-103	Two (2)	Straight Edge	Heavy	Green	Adult Scalp
500-104	Two (2)	Serrated Edge	Light	Yellow	Pediatric Scalp
500-105	Two (2)	Serrated Edge	Medium	Blue	Adult Scalp
500-106	Two (2)	Serrated Edge	Heavy	Green	Adult Scalp

#### SINGLE CARTRIDGE PACK, 15 clips per cartridge

500-107	One (1)	Straight Edge	Light	Yellow	Pediatric Scalp
500-108	One (1)	Straight Edge	Medium	Blue	Adult Scalp
500-109	One (1)	Straight Edge	Heavy	Green	Adult Scalp
500-110	One (1)	Serrated Edge	Light	Yellow	Pediatric Scalp
500-111	One (1)	Serrated Edge	Medium	Blue	Adult Scalp
500-112	One (1)	Serrated Edge	Heavy	Green	Adult Scalp



Reusable stainless steel scalp clip applier/remover.  
Catalog No. 500-135

#### Contact ACRA-CUT:

- To arrange a product trial.
- To place an order.
- For additional information on our complete range of cranial perforators, cutting tools and other instruments.

### FOR MORE INFORMATION



989 MAIN STREET, ACTON, MASSACHUSETTS 01720

Toll free 1-800-ACRA-CUT (800-227-2288) In Massachusetts (978) 263-0250 FAX (978) 263-4102  
www.acracut.com

U.S. Patent No. 5,665,095. Foreign patents pending.

Printed in U.S.A. 501/7.5M © 2001 ACRA-CUT, Inc.

An educational videotape, *Recent Advances in Cranial Perforation*, is now available. Please call for your free copy.